LYNX


INSTRUCTION FOR USE (IFU)

EN 3

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
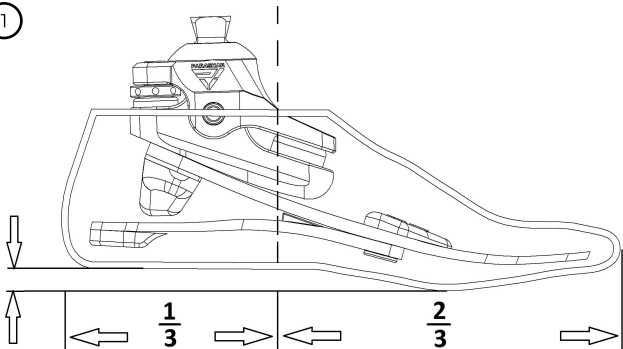
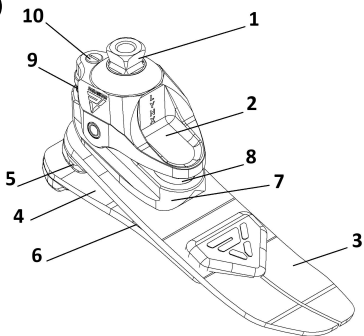
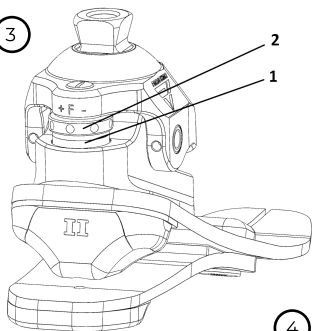
④

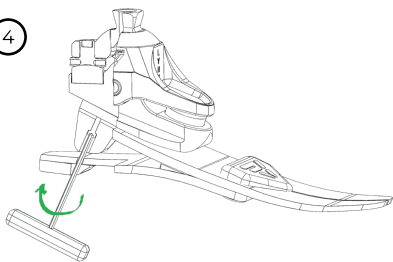⑤

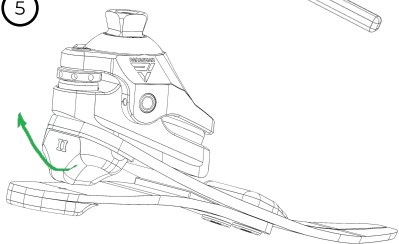INFORMATION**Instructions for use last updated: 2026-01-03**

- ▶ Read this document carefully before using the product and follow the safety warnings.
- ▶ Teach the user how to use the product safely.
- ▶ Contact the manufacturer if you have any questions about the product or have any problems.
- ▶ Report every serious incident related to the product to the manufacturer and the relevant authority in your country. This is especially important when health is deteriorating.
- ▶ Save this document for your records.

The product "Foot LYNX" is hereinafter simply called the product/foot/prosthetic foot.

These instructions for use contain important information regarding the application, adaptation and handling of the product.

Use the product only in accordance with the information contained in the accompanying documents.

2 Product Description

2.1 Function

The LYNX is a low-profile prosthetics foot specially designed to provide anatomical ankle movement and a stable gait under daily and intense stress. Its design is based on the principles of three-axis movement, which allows the foot to naturally respond to surface changes and dynamically adapt in the sagittal, coronal and transverse planes. With internal shock absorbers and a carbon fiber base, the LYNX provides a smooth transition from heel to toe, excellent energy output, and reduced stress on the joints during walking or physical activity. This design allows for a balance between flexibility and stability, making the LYNX a comfortable choice for users with different levels of mobility.

2.2 Combination capabilities

This prosthetic component is compatible with Parashar Industries' modular component system. Compatibility with components from other manufacturers that have compatible modular connectors has not been tested.

3 Intended Use

3.1 Indications for use

The product is intended exclusively for exoprosthetics of the lower extremities.

3.2 Terms of Use

The product is designed for everyday use and should not be used for unusual activities (e.g. extreme sports: rock climbing, parachuting, paragliding, etc.).

Permissible environmental conditions are described in the technical data. Our components work optimally in combination with the corresponding components according to body weight and mobility level according to the K-level classification, which have suitable modular connectors. The product is recommended for mobility levels 3 (free walking outdoors) and 4 (free walking outdoors with particularly high requirements).

The allowed maximum body weight is 125 kg.

3.3 Indications

- For patients with amputation of the lower leg, thigh, dismemberment in the hip joint;
- For unilateral or bilateral amputation.

3.4 Contraindications

• All conditions that contradict or exceed the specifications specified in the "Safety" and "Indications for Use" sections.

3.5 Qualifications

A patient can only be tried on with the product by a certified specialist in orthotics and prosthetics.

4 Safety

4.1 Explanation of Warning Symbols

Symbol	Meaning
WARNINGS	Warning of possible serious risks of accident or injury.
CAUTION	Warning of possible risks of accident or injury.
REMARKS	Warning of possible technical damage.

4.2 Safety Instruction Structure

WARNINGS

The title describes the source and/or type of hazard. The introduction describes the consequences of not following safety instructions.

4.3 General Safety Instructions

WARNINGS

Failure to comply with safety warnings

Injury/damage to the product due to the use of the product in certain situations.

- Follow the safety warnings and precautions specified in this document.

WARNINGS

Fitting in the presence of children

Trauma due to swallowing small parts.

- Never leave children unattended during the fitting.

CAUTION

Self-intervention in the product

Falling due to failure of loaded components or product failure.

- Any manipulation of the product, except for those described in these instructions, is prohibited;
- Only the Parashar Industries service center has the right to disassemble the product.

4.4 Alignment/Adjustment Information

CAUTION

Incorrect alignment or assembly

Falls due to damage to the components of the prosthesis.

- Follow the alignment and assembly instructions.

CAUTION

Failure to comply with the tightening torques of screw connections

Falls due to failure of loaded components due to loose screws.

- Compliance with the specified tightening torques is mandatory for safe operation;
- The specified tightening torques must not be exceeded.

CAUTION

Incorrectly fixed screws

Falls due to breakage of loaded components due to loose screw connections.

- After completing all settings, the screws connecting the foot pyramid to the receiving pyramid of the adapter must be fixed with glue before tightening to the specified torque.

CAUTION

Use of incompatible prosthetic components

Falls due to unexpected product behavior or failure of loaded components.

- Use the product only in combination with components approved by Parashar Industries (see instructions and catalogs);
- Pay attention to prohibited combinations in the "Combination Options" section;
- If the product is used in water, make sure that each component of the prosthesis is waterproof.

CAUTION

Improper assembly of screw connections

Falling due to breakage or loosening of screw connections.

- Clean the threads before each installation;
- Follow the instructions for fixing screws and using the correct length.

4.5 Patient Information

CAUTION

Penetration of dirt and moisture into the product

Falling due to unexpected product behavior due to malfunction.

- Do not allow solid particles or liquids to enter the product.
- Sand or debris in the product may be a source of noise.

CAUTION

Overload due to unusual activities

Fall due to failure of loaded components.

- The product is designed for everyday use and should not be used for extreme activities.
- Careful handling of the product and its components not only prolongs the service life, but also ensures your personal safety!
- If the product has been subjected to extreme loads (e.g. falls), it must be immediately checked for damage by a prosthetist. If necessary, the product will be sent to a Parashar Industries service center.

REMARKS

Improper product care

Damage to the product due to the use of improper cleaning products and/or disinfectants.

- Clean the product only with a damp cloth and mild soap.
- Disinfect the product with isopropyl alcohol as needed.
- Do not use disinfectants based on hydrogen peroxide and silver salts.
- Use only fresh water with a temperature not exceeding 65 °C for cleaning.

5 Package Contents and Accessories

5.1 Package Contents

1. 1 pc. foot LYNX;
2. 1 pc. cosmetic shell;
3. 1 pc. protective sock;
4. 1 pc. Instructions for use (for qualified personnel).

5.2 Accessories replaceable stiffness dampers (supplied separately).

5.3 Spare parts (supplied separately).

6 Preparing the Product for Use

6.1 Required tools

1. Torque wrench up to 30 Nm;
2. Adhesive Loctite 243.

6.2 Foot mounting

6.2.1 Basic assembly

The correct positioning of the prosthetic foot is key to an efficient gait and even distribution of the load. In Fig. (1) depicts the recommended position of the foot relative to the vertical load line that passes through the patient's center of gravity (anatomical axis).

Basic installation rules:

Foot position

The axis of the prosthetic foot should be located so that the load line crosses the foot in a ratio of 1/3 to 2/3:

1/3 of the length of the foot behind the line

2/3 — ahead

Vertical distance

The heel of the foot should be located about 10 mm above the supporting surface at rest (without load). This ensures the correct starting position when walking.

Component orientation

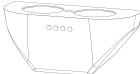
The tilt angle adjuster (if any) must be adjusted so that during dynamic loading, the center of pressure moves smoothly from the heel to the forefoot.

The key components of the product are shown in Fig. ②:

1. Connecting pyramid;
2. Foot bracket top;
3. Toe spring;
4. Heel spring;
5. Heel damper;
6. Interplate damper;
7. Foot bracket bottom;
8. Front damper;
9. Roll damper;
10. Roll damper adjuster.

6.2.2 Adjusting the Elasticity of the Foot

To ensure comfortable and efficient use of the foot, it is possible to adjust the stiffness of the heel part in accordance with the patient's body weight. For this purpose, four variable damper options have been developed, each designed for a specific range of user weight. Replacing the damper allows you to precisely adapt the level of shock absorption and ensure proper stability while walking. Installation of the appropriate damper option is carried out in accordance with the recommendations given in the table. A mismatch between the type of damper and the user's weight may result in a decrease in the efficiency or comfort of the product.

Damper type	Patient's weight, kg	Mobility level
	60-80	K3-K4
	70-90	
	80-100	
	100-125	

6.2.3 Replacing the heel damper

Before starting to dismantle the damper, it is necessary to carefully remove the foot from the cosmetic shell to ensure free access to the internal structural elements.

Using a 2.5 mm hex wrench, unscrew the locking screw that holds the damper in the set position Fig. ③.

After loosening and completely removing the screw, carefully remove the damper from the seat (Fig. (4)), avoiding applying excessive force so as not to damage adjacent components of the foot.

6.2.4 Damper System Adjustment

In order to ensure an anatomically approximate, stable and comfortable gait, a special damper system has been implemented in the design of the artificial foot. It is based on the principles of three-axis movement, which allows the foot to naturally respond to changes in surface, adapt to different walking conditions and absorb shock loads when in contact with the supporting surface. Thanks to this system, dynamic balancing and user support are achieved at every stage of the step.

The level of stiffness of the damper system can be adjusted by the prosthetist by rotating a special regulator located in position 2 (diagram (5)). To increase rigidity and increase resistance, rotate the regulator in the direction marked as "F +". To reduce stiffness and achieve a softer shock-absorbing effect, it is recommended to adjust gradually, controlling changes in sensations while walking.

7 Maintenance

Annual safety inspections are recommended to ensure patient safety and product performance.

8 Cleaning

1. Clean the product with a damp cloth and soap.
2. Rinse with clean fresh water.
3. Dry with a soft cloth.
4. **NOTE!** Do not store the product wet. Allow it to air dry.

9 Disposal

In some countries, it is forbidden to dispose of these products with household waste. Improper disposal can be harmful to health and the environment. Follow the instructions of the national authority for return and collection.

10 Legal Information

All legal terms are subject to the laws of the country of use.

10.1 Liability

The manufacturer is only responsible if the product is used in accordance with these instructions. The manufacturer is not responsible for damage caused by non-compliance with the instructions, improper use, combination with components from other manufacturers, or unauthorized intervention.

10.2 Trademarks

All product names are trademarks owned by Parashar Industries.

10.3 CE Compliance

Parashar Industries declares that the product complies with current European requirements for medical devices.

10.4 Compliance with the Technical Regulations of Ukraine

The manufacturer Parashar Industries confirms that the product meets the requirements of the Technical Regulation on Medical Devices, approved by the Resolution of the Cabinet of Ministers of Ukraine No. 753 dated October 2, 2013, provided that the user complies with the established requirements for transportation, storage, installation and operation of the product.

10.5 Warranty

The manufacturer provides a warranty for this product from the date of purchase.

The warranty covers failures caused by confirmed defects in material, manufacturing defects or design defects that have been notified to the manufacturer during the warranty period.

Detailed information on the terms of the warranty can be obtained from the relevant representative of the manufacturer or from the company supplying the products.

11 Technical Data

Environmental conditions

Permissible conditions for using the product:

Temperature range of use from -10°C to +60°C

Storage and transportation conditions: from -10°C to +60°C

Fresh water, soapy water, urine (needs to be washed)

Swimming pool (needs flushing)

Sea and chlorinated water (requires flushing)

Unacceptable conditions for using the product:

Mechanical vibration or mechanical shock

Ingress of dust, sand, hygroscopic parts (e.g. talc)

Acidic and alkaline solutions

Solvent-containing cleaners

Product characteristics

Parameter	LYNX
Weight with cosmetic shell [g]	910–970
Proximal connection	Aligning pyramid
Max. Body weight [kg]	125
Mobility level	K3-K4
System Height [mm]	80-85
Material	Composite
Service life [years]	3

12 Appendices

12.1 Symbols Used

 Serial number	 Keep out of direct sunlight
 Manufacturer	 Protect from moisture
 Date of manufacture	 Medical device
 See instructions for use	 Conformity mark without an authority code
 Multiple uses, but for one patient	 Keep out of direct sunlight
 Declaration of conformity to European directives	

Parashar Industries LLC
12 Makarivska Street, Petropavlivska
Borshchahivka village, Bucha District,
Kyiv Region, 08129, Ukraine
tel.: +38 (098) 227 2659
info@parasind.com
www.parasind.com

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This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

[illegible]

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.