# X-Lock

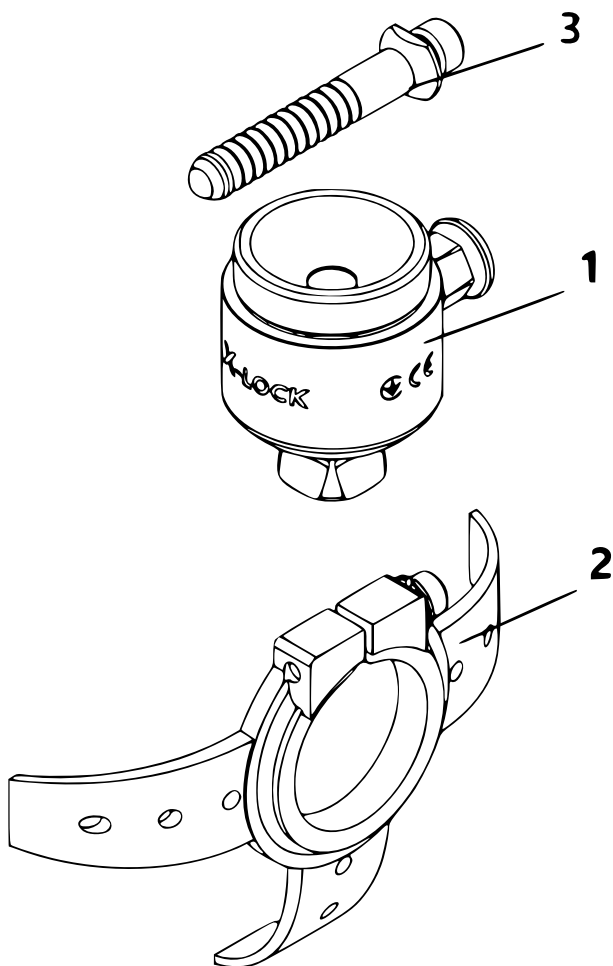
CE

## INSTRUCTION FOR USE (IFU)

EN ..... 3

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## INFORMATION

### Instructions for use last updated: 2026-01-03

- Read this document carefully before using the product and follow the safety warnings.
- Teach the user how to use the product safely.
- Contact the manufacturer if you have any questions about the product or have any problems.
- Report every serious incident related to the product to the manufacturer and the relevant authority in your country. This is especially important when health is deteriorating.
- Save this document for your records.

The product "Liner Lock X-LOCK" is hereinafter simply called the product/liner.

These instructions for use contain important information regarding the application, adaptation and handling of the product.

Use the product only in accordance with the information contained in the accompanying documents.

## 2 Product Description

### 2.1 Function

The X-LOCK liner lock is designed to fix the liner in a lower leg or thigh prosthesis. The lock is installed by lamination in the distal part of the stump receiving sleeve.

### 2.2 Combination capabilities

This prosthetic component is compatible with Parashar Industries' modular component system. Compatibility with components from other manufacturers that have compatible modular connectors has not been tested.

## 3 Intended Use

### 3.1 Indications for use

The product is intended exclusively for exoprosthetics of the lower extremities.

### 3.2 Terms of Use

The product is designed for everyday use and should not be used for unusual activities (e.g. extreme sports: rock climbing, parachuting, paragliding, etc.).

Permissible environmental conditions are described in the technical data.

Our components work optimally in combination with the corresponding components according to body weight and mobility level according to the K-level classification, which have suitable modular connectors.

The product is recommended for mobility levels 1 (indoor walking), 2 (restricted outdoor walking), 3 (outdoor free walking) and 4 (outdoor free walking with particularly high requirements).

**The allowed maximum body weight is 125 kg.**

### 3.3 Indications

- For patients with amputation of the lower leg, thigh;
- For unilateral or bilateral amputation.

### 3.4 Contraindications

- All conditions that contradict or exceed the specifications specified in the "Safety" and "Indications for Use" sections.

### 3.5 Qualifications

A patient can only be tried on with the product by a certified specialist in orthotics and prosthetics.

## 4 Safety

### 4.1 Explanation of Warning Symbols

| Symbol          | Meaning                                                  |
|-----------------|----------------------------------------------------------|
| <b>WARNINGS</b> | Warning of possible serious risks of accident or injury. |
| <b>CAUTION</b>  | Warning of possible risks of accident or injury.         |
| <b>REMARKS</b>  | Warning of possible technical damage.                    |

## 4.2 Safety Instruction Structure

### **WARNINGS**

The title describes the source and/or type of hazard. The introduction describes the consequences of not following safety instructions.

## 4.3 General Safety Instructions

### **WARNINGS**

#### **Failure to comply with safety warnings**

Injury/damage to the product due to the use of the product in certain situations.

- Follow the safety warnings and precautions specified in this document.

### **WARNINGS**

#### **Fitting in the presence of children**

Trauma due to swallowing small parts.

- Never leave children unattended during the fitting.

### **CAUTION**

#### **Self-intervention in the product**

Falling due to failure of loaded components or product failure.

- Any manipulation of the product, except for those described in these instructions, is prohibited;
- Only the Parashar Industries service center has the right to disassemble the product.

## 4.4 Alignment/Adjustment Information

### **CAUTION**

#### **Incorrect alignment or assembly**

Falls due to damage to the components of the prosthesis.

- Follow the alignment and assembly instructions.

**CAUTION****Mechanical damage to the product**

Risk of injury due to alteration or loss of function

- The product must be handled with care.
- A damaged product should be checked for functionality and usability.
- Do not use the product in case of change or loss of function (see Fig. "Signs of change or loss of function during operation" in this section).
- If necessary, take appropriate measures (e.g. repair, replacement, inspection by the manufacturer's service department, etc.).

**CAUTION****Failure to comply with the tightening torques of screw connections**

Falls due to failure of loaded components due to loose screws.

- Compliance with the specified tightening torques is mandatory for safe operation;
- The specified tightening torques must not be exceeded.

**CAUTION****Exceeding the service life**

Risk of injury due to changes or loss of function, as well as damage to the product

- It should be noted that the period of operation has not been exceeded.

**CAUTION****Use of incompatible prosthetic components**

Falls due to unexpected product behavior or failure of loaded components.

- Use the product only in combination with components approved by Parashar Industries (see instructions and catalogs);
- Pay attention to prohibited combinations in the "Combination Options" section;
- If the product is used in water, make sure that each component of the prosthesis is waterproof.

**CAUTION****Incorrect installation of threaded connections**

Incorrect installation of threaded connections

Risk of injury due to breakage or loosening of threaded connections

- Clean the threads each time before installation.
- Observe the tightening torques during installation.

**CAUTION****Improper handling of sleeve adapter petals**

Risk of injury due to the destruction of the petals

- Only wiring the steel paddles of the adapter
- Avoid too much and frequent wiring of petals
- For wiring, you should use a suitable tool for such work.

**Signs of change or loss of function during operation**

Changes in functions can manifest themselves, for example, in the form of changes in the gait pattern, changes in the placement of the prosthesis components in relation to each other, as well as the presence of noise.

**4.5 Patient Information****CAUTION****Penetration of dirt and moisture into the product**

Falling due to unexpected product behavior due to malfunction.

- Do not allow solid particles or liquids to enter the product.

**CAUTION****Overload due to unusual activities**

Fall due to failure of loaded components.

- The product is designed for everyday use and should not be used for extreme activities.
- Careful handling of the product and its components not only prolongs the service life, but also ensures your personal safety!
- If the product has been subjected to extreme loads (e.g. falls), it must be checked immediately for damage by a prosthetist. If necessary, the product will be sent to the Parashar Industries service center.

## **REMARKS**

### **Improper product care**

Damage to the product due to the use of improper cleaning products and/or disinfectants.

- ▶ Clean the product only with a damp cloth and mild soap.
- ▶ Disinfect the product with isopropyl alcohol as needed.
- ▶ Do not use disinfectants based on hydrogen peroxide and silver salts.
- ▶ Use only fresh water with a temperature not exceeding 65 °C for cleaning.

## **5 Package Contents and Accessories**

### **5.1 Package Contents**

Figure 1

1. 1 x X-LOCK liner lock;
2. Bottom bracket
3. 1pc. pin short/long;
4. 1 pc. Instructions for use (for qualified personnel).

**5.2 Accessories** (supplied separately).

**5.3 Spare parts** (supplied separately).

## **6 Preparing the Product for Use**

### **6.1 Required tools**

1. Torque wrench up to 30 Nm;
2. Adhesive Loctite 243.

### **6.2 Mounting the Adapter**

#### **6.2.1 X-LOCK Liner Lock**

The product is mounted to the culti-receiving sleeve in the process of its lamination using reinforcing materials (fiberglass, carbon fiber, basalt fiber). The number of layers of lamination should correspond to the technological process in accordance with the activity and weight of the patient.

## **CAUTION**

### **Insufficient screwing depth into the lamination anchor**

Fall due to failure of loaded components.

- ▶ After the rotary adapter is fully screwed into the anchor thread, unscrew it no more than 3/4 turn.

## **INFORMATION**

The screw is tightened to a maximum of 8 Nm.

### **During installation, it is imperative:**

1. Contact with the frame or hydraulic block of the knee assembly with maximum knee flexion should be excluded.

### **Fitting:**

1. Tighten the M5×30 screw on the adapter paddle with a torque wrench to 8 Nm.

### **Final consolidation:**

1. Tighten the M5×30 screws on the adapter petals (bottom bracket Fig.1) with a torque wrench up to 8 Nm.

## **7 Maintenance**

Annual safety inspections are recommended to ensure patient safety and product performance.

## **8 Cleaning**

1. Clean the product with a damp cloth and soap.
2. Rinse with clean fresh water.
3. Dry with a soft cloth.
4. **NOTE!** Do not store the product wet. Allow it to air dry.

## **9 Disposal**

In some countries, it is forbidden to dispose of these products with household waste. Improper disposal can be harmful to health and the environment. Follow the instructions of the national authority for return and collection.

## **10 Legal Information**

All legal terms are subject to the laws of the country of use.

### **10.1 Liability**

The manufacturer is only responsible if the product is used in accordance with these instructions. The manufacturer is not responsible for damage caused by non-compliance with the instructions, improper use or unauthorized intervention.

### **10.2 Trademarks**

All product names are trademarks owned by Parashar Industries.

### **10.3 CE Compliance**

Parashar Industries declares that the product complies with current European requirements for medical devices.

### **10.4 Compliance with the Technical Regulations of Ukraine**

The manufacturer Parashar Industries confirms that the product meets the requirements of the Technical Regulation on Medical Devices, approved by the Resolution of the Cabinet of Ministers of Ukraine No. 753 dated October 2, 2013, provided that the user complies with the established requirements for transportation, storage, installation and operation of the product.

### **10.5 Warranty**

The manufacturer provides a warranty for this product from the date of purchase.

The warranty covers failures caused by confirmed defects in material, manufacturing defects or design defects that have been notified to the manufacturer during the warranty period.

Detailed information on the terms of the warranty can be obtained from the relevant representative of the manufacturer or from the company supplying the products.

## **11 Technical Data**

### **Environmental conditions**

#### **Permissible conditions for using the product:**

Temperature range of use from -10°C to +60°C

Storage and transportation conditions: from -10°C to +60°C

Fresh water, soapy water, urine (needs to be washed)

Swimming pool (needs flushing)

Sea and chlorinated water (requires flushing)

#### **Unacceptable conditions for using the product:**

Mechanical vibration or mechanical shock

Ingress of dust, sand, hygroscopic parts (e.g. talc)

Acidic and alkaline solutions

Solvent-containing cleaners

## Product characteristics

| Parameter             | X-LOCK liner lock     |
|-----------------------|-----------------------|
| Weight [g]            | 294                   |
| Proximal connection   | Threaded M36          |
| Distal junction       | Justification pyramid |
| Max. Body weight [kg] | 125                   |
| System Height [mm]    | 125                   |
| Material              | Aluminum alloy, steel |
| Service life [years]  | 3                     |

## 12 Appendices

### 12.1 Symbols Used

|                                                                                                                                   |                                                                                                                             |
|-----------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------|
|  Batch number                                     |  Keep out of direct sunlight               |
|  Manufacturer                                     |  Protect from moisture                     |
|  Date of manufacture                              |  Medical device                            |
|  See instructions for use                         |  Conformity mark without an authority code |
|  Multiple uses, but for one patient               |  Keep out of direct sunlight               |
|  Declaration of conformity to European directives |                                                                                                                             |

**Parashar Industries LLC**

12 Makarivska Street, Petropavlivska  
Borshchahivka village, Bucha District,  
Kyiv Region, 08129, Ukraine

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